


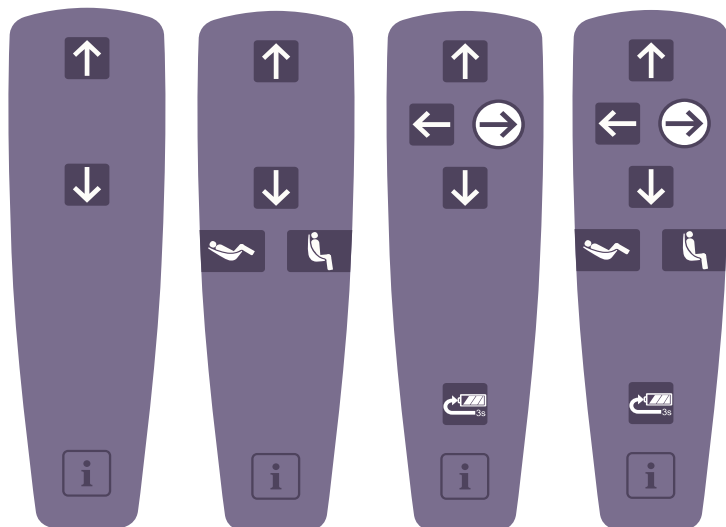







# Maxi Sky 2








## Hand control

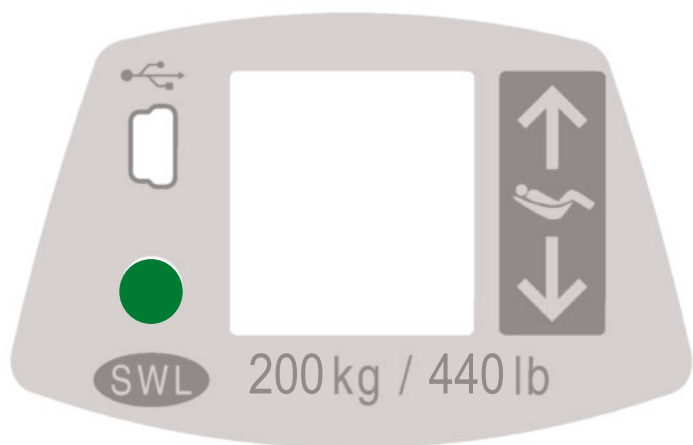

-  **Up Button**
-  **Down Button**
-  **Right Button**
-  **Left button**
-  **Reclined Button** (Only Powered DPS)
-  **Sit Button** (Only Powered DPS)
-  **RTC** (Return to Charge)
-  **RTH** (Return to Home) Button
-  **Programming / Information Button**

## Control panel



-  **Auxiliary Up-Down Button**  
The up and down buttons are used to complete a transfer if a problem occurs with the hand control during the operation.
-  **Status LED**  
The LED is a complement to the display to draw attention that a message is being displayed on the screen.
-  **SWL**  
The label indicates the maximum lifting capacity of the lift.
-  **USB Access Port**  
The USB port is intended for MAXISMART users only.

## Display symbols



### Battery Charge Level



100%



75%

Normal operating range



50%



25% Low

Accompanied with a short beep when a button is pressed



< 5%

Critical

Icons alternate with two short beeps every 10 seconds



### Enhanced Charging System (ECS) Indicator

Displayed when charging units with ECS.



### Return To Charge (RTC)

Indicates that the RTC function is activated by the hand control.



### Return to Home (RTH)

Indicates that the RTH function is activated by the hand control.

PPP



### Pre-Programmed Position (PPP)

Indicates that the PPP function is activated by the hand control.



### Over Duty Cycle

Flashes when the lift is used for a long period without allowing it to cool down.

The overheat protection is activated when a function exceeds its duty cycle, is becomes disabled while all other functions remain active.

The red light stays on during the cooling period and a single beep is heard.

The symbol disappears when the function has cooled down and it is ready for use again.



### General Malfunction (Warning symbol accompanied with a numeric character)

Flashes when the lift encounters a malfunction.

The lift needs to be inspected or repaired by a qualified service technician.

The red light stays on and a single beep is heard.



ABC

### Maintenance Required (Wrench symbol accompanied with a letter character)

Shows for 7 seconds (accompanied with 2 beeps) when the lift goes from sleep mode and flashes each 5 seconds to draw user's attention.

The unit can be used but maintenance shall be done as soon as possible by an Arjo technician.





### Overweight

Flashes when a transfer is attempted with a load exceeding the lift's capacity.



The red light stays on (accompanied with 4 rapid beeps) until the patient is lowered.

The function is disabled while all other functions remain active.

## Spreader Bar and Scale Attachment

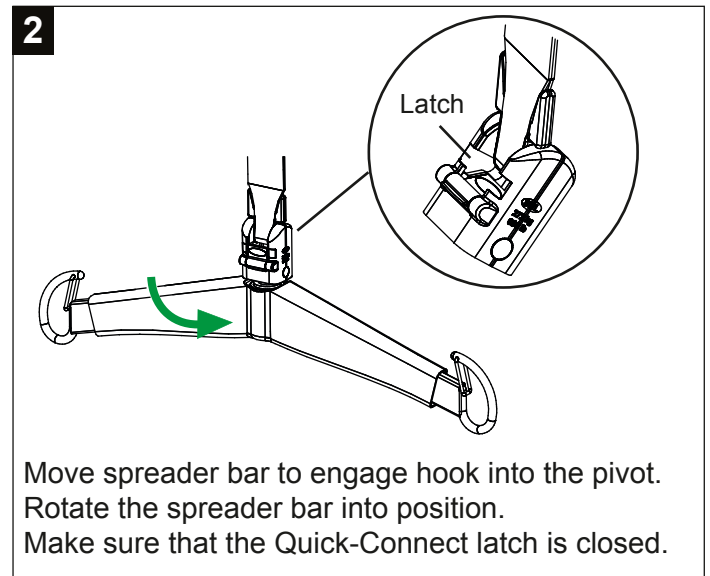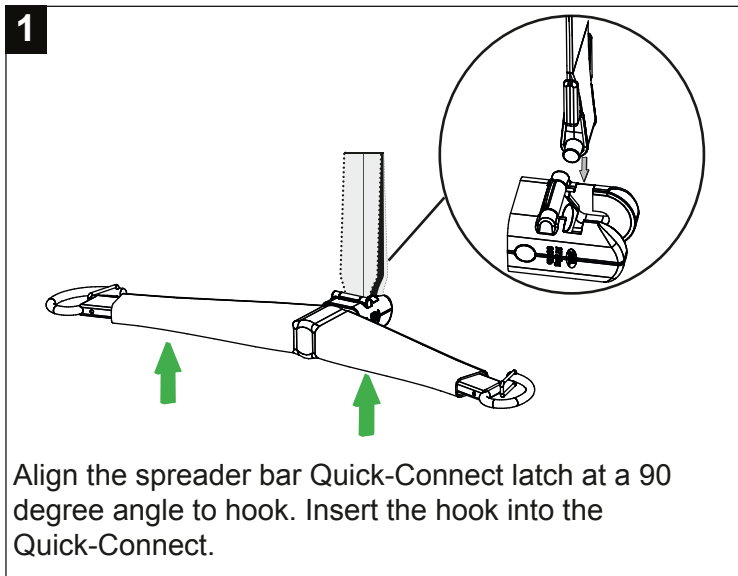

## Spreader Bar and Scale Removal

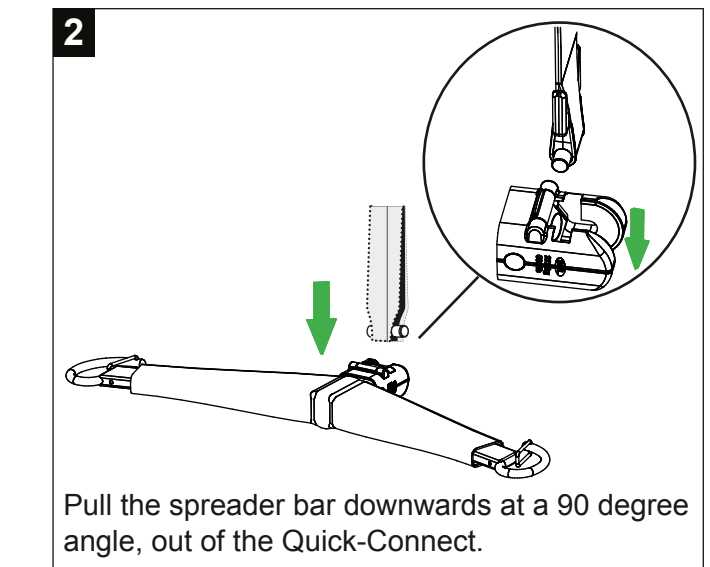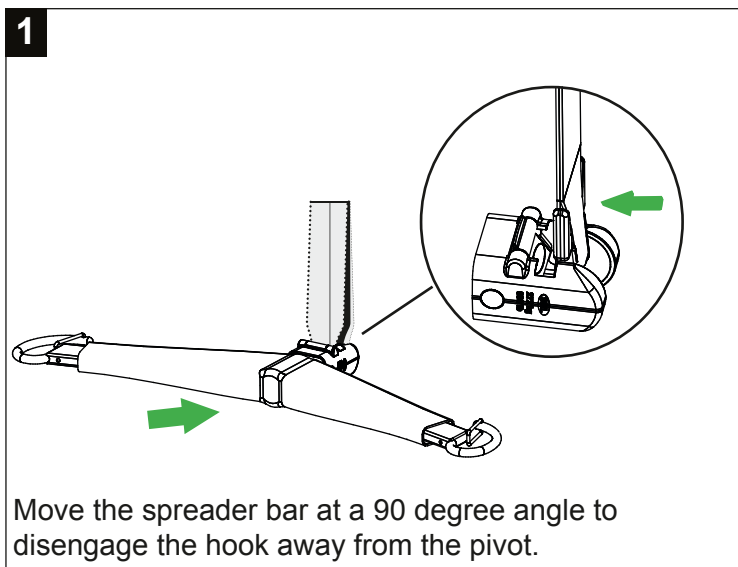

## Spreader Bar and Scale Attachment or Removal (Basic Ceiling Lifter model only)

